



Designed with patients in mind

With breath actuated delivery, aerosol is **only** produced when you breathe in. This means medication is not wasted between breaths or if you need to take a break during your treatment. Medication is delivered **deep into the lungs** where it's most needed.¹

Breath Actuated Delivery

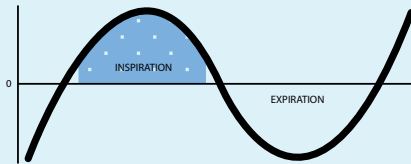

Conventional Continuous Nebulizers

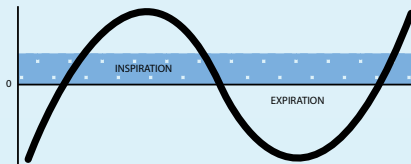

Paired with **Ombra*** compressors for a complete delivery system at home



Can be combined with the **Aerobika*** OPEP device to reduce overall treatment time. Speak to your healthcare provider for more information.



OPEP = oscillating positive expiratory pressure

AeroEclipse* XL


BAN* Nebulizer

OMBRA*


Compressors

HOW TO USE


Nebulizer Top

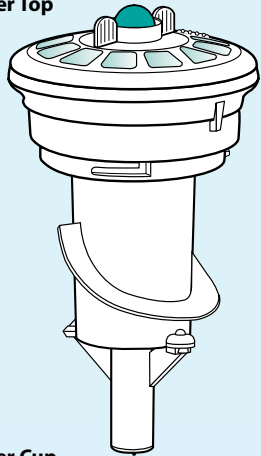

Nebulizer Cup

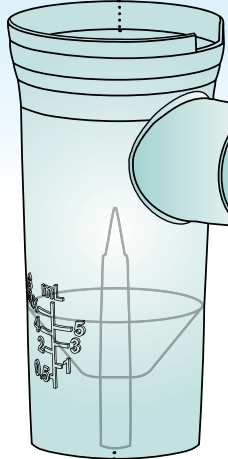


EZ Twist Tubing

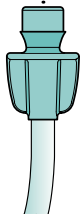

- 1 Place your compressor on a clean, flat surface.
- 2 Inspect your nebulizer and ensure the green exhalation valve on the mouthpiece is facing down. Unscrew and remove the nebulizer top. Carefully place prescribed medication into the nebulizer cup and replace the nebulizer top. Keep the nebulizer in an upright position to prevent spills. The nebulizer may be placed in the cup holder on the compressor.
- 3 Securely connect the tubing to the bottom of the nebulizer and to the air outlet on your compressor.
- 4 In a comfortably seated position, switch on your compressor. Place the mouthpiece in your mouth, ensuring your lips are sealed around the edge but aren't covering the green exhalation valve. If you are using a mask, place securely over the nose and mouth, ensuring a good seal.
- 5 Breathe in slowly and deeply, then exhale normally through the device. Continue taking slow, deep breaths until the nebulizer begins to make a 'sputter' noise. Switch off your compressor when your treatment is complete.

(Refer to the [instructions for use](#) for detailed guidance.)


Mouthpiece

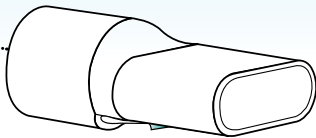

Air Outlet

Nebulizer Cup Holder

On/Off Switch

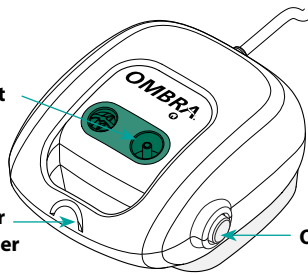

Ombra* Table Top Compressor shown


HOW TO CLEAN



WASH
in soapy water




RINSE
in clean water

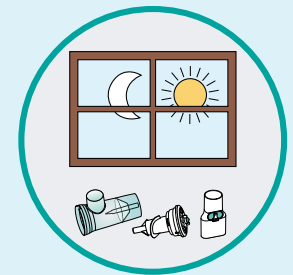

AIR DRY
thoroughly

Notes:

DISHWASHER SAFE
top rack in basket

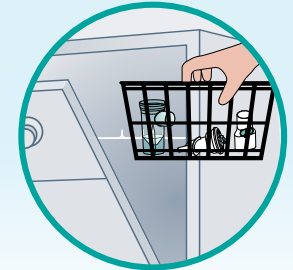

The **AEROECLIPSE* XL BAN*** Nebulizer is a single-patient use device, reusable for 6 months.

HOW TO DISINFECT

BOILING:

The nebulizer (excluding tubing) may be boiled in water for 5-15 minutes.

ELECTRONIC STEAM DISINFECTOR:

The nebulizer (excluding tubing) may be disinfected using an electronic steam sterilizer.

Refer to the [instructions for use](#) for additional information.

MICROWAVE STEAM BAG:

The nebulizer (excluding tubing) may be disinfected in the microwave in a steam cleaning bag.

70% ISOPROPYL ALCOHOL:

The nebulizer (excluding tubing) may be soaked in 70% isopropyl alcohol for 5 minutes.

aeroclipse.com



LEARN MORE



Designed and
Developed in Canada

Manufactured in Canada
with Canadian and imported parts.

