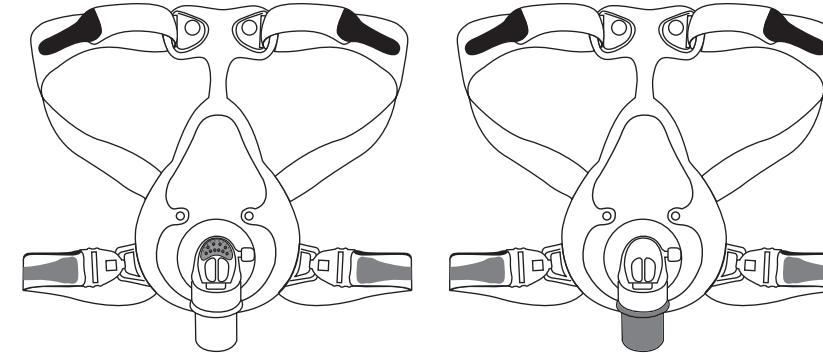
REF NIV040S, NIV040M, NIV040L, NIV043S, NIV043M, NIV043L

AirLife™

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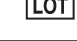
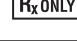
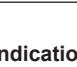
Non Invasive Ventilation (NIV) Vented and Non-Vented Full Face Mask with Anti-Asphyxiation Valve

For single patient use

Figure 1	Figure 2
	
Place the headgear over your head and adjust both upper headgear straps to a comfortable tension.	Hook the plastic buckle onto the mask frame.
Figure 3	Figure 4
	
Adjust both upper headgear straps to a comfortable tension.	Adjust both lower headgear straps to a comfortable tension.
Figure 5	
	
Making sure the overhead strap is in proper position to avoid slipping down.	
*To remove the mask and headgear, simply unbuckle both buckles from the mask frame and pull the upper head strap away from the head.	

Symbols

	Catalogue number		Manufacturer
	Lot Number		U.S. Federal law restricts this device to sales by or on the order of a physician
	Not made with Phthalates (DEHP)		Quantity
	Not made with natural rubber latex		Exp. Date
	Consult instructions for use		Date of Manufacture
	Authorized Representative in the European Community		Single Use

Indications for Use

AirLife Vented and Non-Vented Full Face Mask is a patient interface for use as an accessory to a respiratory therapy device, i.e. ventilator that has adequate alarms and safety systems for ventilator failure, providing noninvasive positive pressure respiratory therapy. The mask is for single patient use with spontaneously breathing adult (> 30 kg) patients with respiratory insufficiency or respiratory failure who are suitable for noninvasive pressure support ventilation in home, hospital and institutional environments.

Features

- One piece molded silicone rubber; small, light and comfortable.
- Clear mask allows a good view of mouth and nasal area.
- Low dead space results in low CO₂ build up.
- Dual 360 degree swivels at the mask attachment and at the standard 22 mm tubing attachment point.
- Headgear with Velcro™ fasteners provide maximum versatility and movement.

Technical Specifications

- Operating pressure range 4-20 cmH₂O
- Interface connections ISO 5356-1 Conical Connectors
 - Vented mask: 22 mm O.D.
 - Non-vented mask: 22 mm I.D.
- Unintentional mask leak, excluding leak between face and mask, is less than 5 LPM at 10 cmH₂O

Small—Dead space = 284 ml

Medium—Dead space = 303 ml

Large—Dead space = 355 ml

Contraindications

This mask may not be suitable for patients with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex and hiatal hernia. It should not be used if the patient is unresponsive, obtunded, unable to remove the mask, or uncooperative.

Care Instructions

Clean the NIV Mask using warm water with a mild soap. Follow your facility's procedures for disposing of medical devices.

The use of bleach, chlorine, alcohol or aromatic-based solutions may cause hardening of the materials of construction and reduce the life of the product.

Warnings

At low NIV pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing and some rebreathing may occur.

- This mask should only be used with ventilation systems recommended by your physician or respiratory therapist.
- Do not use this mask unless the ventilation system is turned on and operating properly.
- Never block the mask's vent holes or exhalation port.
- If oxygen is used with the ventilation system, the oxygen flow must be turned off when the ventilator machine is not in operation.
- Do not smoke if oxygen is in use.
- Reuse may result in transmission of infectious substances, interruption to treatment, serious harm or death.
- Contact a health care professional if chest discomfort, shortness of breath, stomach distention, belching, or severe headache are experienced.
- Must only be used by persons who have received adequate training and instruction from qualified medical personnel.

Note: Ventilation systems are intended to be used with special masks with ventholes in either the connectors or the mask that allows continuous flow out of the mask. When the ventilator machine is turned on and functions properly, new air from the ventilator machine flushes the exhaled air out through the mask's exhalation port. However, when the ventilator machine is not operating, enough fresh air will not be provided through the mask and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.

Note: When the NIV device is not in operation and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the ventilator machines enclosure and create a risk of fire.

Note: At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary depending on the pressure setting, patient pattern and leak rate.

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© 2018 Vyair. Vyair, the Vyair logo and AirLife are trademarks or registered trademarks of Vyair Medical, Inc., or one of its affiliates.