Clear Anatomical Face Mask

Single-Patient-Use

INDICATIONS FOR USE:

The intended use of the Clear Anatomical Face Mask is to direct gas to the upper airway.

WARNINGS:

- This mask is non-conductive and should not be used in the presence of flammable anesthetics.
- Do not clean, soak, rinse or sterilize. To do so may result in harmful residuals or have adverse side effects on this product.
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
- If the patient experiences skin irritation, consult physician.
- This Mask is not suitable for providing mechanical life support ventilation.
- In the USA, federal law restricts this Mask to sale by, or on the order of a physician. (Rx ONLY)
- This device is assembled with PVC components which include low levels of a phthalate as indicated. Consideration should be used when treating children or pregnant or nursing women.

CAUTIONS:

- During use, the patient's condition must be continuously monitored.

DIRECTIONS FOR USE:

- Connect to breathing circuit or resuscitation device with appropriate fitting.
- Inspect all fittings to ensure secure connections.
- If the mask contains an inflation valve, inflate or deflate the air cushion using a needleless syringe until proper patient fit is accomplished.
- Secure by hand or use a mask strap to hold in place.
- Dispose of according to facility protocol.


Specifications:

The MERCURY MEDICAL® Clear Anatomical Face Mask was designed to meet ISO 5356-1 requirements.

| | | |
|------------|----------------------------|------------|
| Port Size: | Adult Large (136mm) | 22 mm I.D. |
| | Adult Small (127mm) | 22 mm I.D. |
| | Child (101mm) | 22 mm I.D. |
| | Pediatric (89mm) | 22 mm I.D. |
| | Infant (72mm) | 15 mm O.D. |
| | Neonatal (61mm) | 15 mm O.D. |

MR CONDITIONAL:

Static magnetic field of 3.0 Tesla or less
Spatial Gradient of 3000 gauss/cm or less

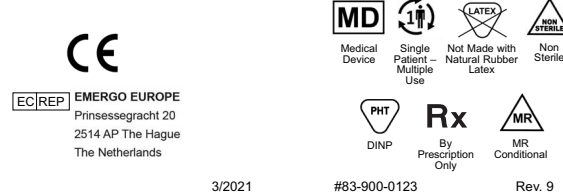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