en	Instructions for use/Technical description Instruments for application in the central nervous system
USA	Note for U.S. users This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com . If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.
de	Gebrauchsanweisung/Technische Beschreibung Instrumente für den Einsatz im zentralen Nervensystem
fr	Mode d'emploi/Description technique Instruments pour applications sur le système nerveux central
es	Instrucciones de manejo/Descripción técnica Instrumentos para el sistema nervioso central
it	Istruzioni per l'uso/Descrizione tecnica Strumenti per sistema nervoso centrale
pt	Instruções de utilização/Descrição técnica Instrumentos para utilização a nível do sistema nervoso central
nl	Gebruiksaanwijzing/Technische beschrijving Instrumenten voor gebruik in het centraal zenuwstelsel
da	Brugsanvisning/Teknisk beskrivelse Instrumenter til anvendelse på det centrale nervesystem
sv	Bruksanvisning/Teknisk beskrivning Instrument för användning i det centrala nervsystemet
fi	Käyttöohje/Tekninen kuvaus Keskushermostossa käytettävät instrumentit
lv	Lietošanas instrukcijas/tehniskais apraksts Instrumenti lietošanai centrālajā nervu sistēmā
lt	Naudojimo instrukcija/techninis aprašas Instrumentai, skirti naudoti centrinėje nervų sistemoje
ru	Инструкция по применению/Техническое описание Инструменты для нейрохирургии
cs	Návod k použití/Technický popis Nástroje k použití v centrálním nervovém systému
pl	Instrukcja użytkowania/Opis techniczny Instrumenty do stosowania w centralnym układzie nerwowym
sk	Návod na použitie/Technický opis Nástroje na použitie v centrálnej nervovej sústave
hu	Használati útmutató/Műszaki leírás A központi idegrendszerben történő használatra szolgáló műszerek
sl	Navodila za uporabo/Tehnični opis Instrumenti za uporabo v centralnem živčnem sistemu
hr	Upute za uporabu/Tehnički opis Instrumenti za primjenu u središnjem živčanom sustavu
ro	Manual de utilizare/Descriere tehnică Instrumente pentru utilizare în sistemul nervos central
bg	Упътване за употреба/Техническо описание Инструменти за използване в централната нервна система
tr	Kullanım Kılavuzu/Teknik açıklama Merkezi sinir sisteminde kullanılan aletler
el	Οδηγίες χρήσης/Τεχνική περιγραφή Όργανα για χρήση στο κεντρικό νευρικό σύστημα

Symbols on product and packages

Caution, general warning symbol
Caution, see documentation supplied with the product

1. Intended use

The instruments are used for holding and dissecting in surgical operations on the central nervous system (e.g. at the pituitary or the brain).

2. Indications

Indications, see Intended use.

Note

The manufacturer is not responsible for any use of the product against the specified indications and/or the described applications.

3. Contraindications

None known.

4. Risks and side effects

As part of the legal obligation to provide information, the possible risks and side effects in connection with the use of surgical instruments known to the manufacturer are highlighted below. These are predominantly process-specific, not product-specific, and include unwanted damage to surrounding tissue, such as bleeding, infection, incompatibility of materials, or parts of instruments left unnoticed in the patient.

5. Safe handling and preparation

- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Read, follow, and keep the instructions for use.
- ▶ Use the product only in accordance with its intended use, see Intended use.
- ▶ Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- ▶ Store any new or unused products in a dry, clean, and safe place.
- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Replace any damaged components immediately with original spare parts.
- ▶ To avoid damage to the working end: Carefully insert the product through the working channel (e.g. trocar).

6. Safe operation

WARNING

Risk of injury and/or malfunction!
▶ Always carry out a function check prior to using the product.

WARNING

Risk of injury when using the product beyond the field of view!
▶ Apply the product only under visual control.

7. Validated reprocessing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

The specified chemistry was used for validation.

7.2 General information

Dried and hardened or clotted surgical residues can complicate cleaning or render it ineffective and cause corrosion. Therefore, do not exceed a period of 1h between use and processing, make sure pre-cleaning temperatures are <math><45^{\circ}\text{C}</math> to prevent clotting, and do not use disinfectants that can cause clotting (active ingredient: aldehyde, alcohol).

Excessive neutralizing agents or basic cleaners may result in a chemical attack and/or fading and the laser marking becoming unreadable either visually or by machine.

Residues containing chlorine or chlorides, e.g., in surgical residues, medicines, saline solutions, and in the service water used for cleaning, disinfection, and sterilization, will cause corrosion damage (pitting, stress corrosion) and result damage to metallic products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Perform additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Material damage such as corrosion, cracks, fracturing, premature aging, or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion
- ▶ For detailed information on hygienically safe and material-friendly/careful processing, please refer to www.a-k-l.org Header "AKI-Brochures", "Red brochure".

7.3 Reusable products

There are no known effects of processing resulting in damage to the product.

The product can be re-used up to 200 times with proper care and if it is undamaged and clean. Any additional reuse falls under the responsibility of the user.

Careful visual and functional testing prior to next use is the best way to identify a malfunctioning product.

7.4 Preparations at the place of use

- ▶ If applicable, rinse surfaces that are not accessible to visible inspection (preferably with demineralized water), using a disposable syringe, for example.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Place the wet product in a sealed waste container and take it to be cleaned and disinfected within 1h.

7.5 Cleaning/disinfection

7.5.1 Product-specific safety notes on the reprocessing procedure

DANGER

Danger to the patient!

- ▶ Reprocess the product only with manual pre-cleaning followed by mechanical cleaning.

DANGER

Risk to patient due to cross contamination!

- ▶ Do not clean contaminated products together with uncontaminated products in a tray.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning and disinfecting agents according to the manufacturer's instructions which
 - are approved for (e.g. aluminum, plastics, high-grade steel),
 - do not attack softeners (e.g. in silicone).
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Do not exceed the maximum cleaning temperature of 55 °C.

- ▶ Use suitable cleaning/disinfecting agents if the product is put away in a wet condition. To prevent foam formation and reduced effectiveness of the process chemicals: Prior to mechanical cleaning and disinfection, rinse the product thoroughly with running water.
- ▶ Clean and disinfect microsurgical products mechanically if they can be placed securely in the machine or on the positioning aids.

7.6 Mechanical cleaning/disinfection with manual pre-cleaning

Note

As a matter of principle, the efficacy of the cleaning and disinfection device must be certified (e.g. in accordance with EN ISO 15883).

Note

The cleaning and disinfection machine used for processing must be serviced and checked at regular intervals.

7.6.1 Manual pre-cleaning with a brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Cleaning	RT (cold)	>15	1	D-W	B. Braun Stabimed® fresh
II	Rinsing	RT (cold)	1	-	D-W	-

D-W: Drinking water
RT: Room temperature

Phase I

- ▶ Clean the product in the solution using a suitable cleaning brush until no residue is visible on the surface.
- ▶ If applicable, use a suitable cleaning brush to brush any surfaces that are not visible for at least 1 min.
- ▶ Mobilize non-rigid components, such as set screws and hinges, during cleaning.
- ▶ Then flush these areas thoroughly at least five times with the cleaning disinfectant solution using a disposable syringe (20 ml).

Phase II

- ▶ Rinse/flush the instrument thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws and hinges, during rinsing.

7.6.2 Mechanical alkaline cleaning and thermal disinfection

Type of device: Single-chamber cleaning/disinfection device without ultrasound¹⁾

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemicals
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	DI-W	Dr. Weigert neodisher® SeptoClean 1 % Working solution ²⁾
III	Neutralization	>10/50	2	DI-W	B. Braun Helimatic® Neutralizer C Working solution 0.15 %
IV	Intermediate rinse I	>10/50	1	DI-W	-
V	Intermediate rinse II	>10/50	1	DI-W	-
VI	Thermal disinfection	90/194	5	DI-W	-
VII	Drying	-	-	-	In accordance with the program for the cleaning and disinfecting machine

D-W: Drinking water
DI-W: Deionized water

- ¹⁾ The cleaning and disinfection devices listed below were used to verify cleanability: Miele 7836 CD
²⁾ Prion-deactivating detergent (see Specifications Dr. Weigert neodisher® SeptoClean)

- ▶ Place the product in a tray that is suitable for cleaning (avoiding rinsing blind spots)
- ▶ Place instruments in the tray with their hinges open.
- ▶ Connect the lumens and channels of individual parts directly to the injector unit's special flushing connector.
- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

7.7 Inspection, maintenance and checks

CAUTION

Damage (metal seizure/friction corrosion) to the product caused by insufficient lubrication!

- ▶ Prior to function checks, lubricate moving parts (e.g. joints, pusher components and threaded rods) with maintenance oil suitable for the respective sterilization process (e.g. for steam sterilization: Aesculap STERILIT® I oil spray JG600 or STERILIT® I drip lubricator JG598).

- ▶ Allow the product to cool down to room temperature.
- ▶ After each complete cleaning, disinfecting and drying cycle, check that the product is dry, clean, operational, and free of damage (e.g. broken insulation or corroded, loose, bent, broken, cracked, worn, or fractured components).
- ▶ Dry the product if it is wet or damp.
- ▶ Repeat cleaning and disinfection of products that still show impurities or contamination.
- ▶ Check that the product functions correctly.
- ▶ Immediately put aside damaged or inoperative products and send them to Aesculap Technical Service, see Technical Service.

7.8 Packaging

- ▶ Appropriately protect products with fine working tips.
- ▶ Store products with ratchet locks fully opened or locked no further than in the first notch.
- ▶ Place the product in its holder or on a suitable tray. Ensure that all cutting edges are protected.
- ▶ Pack trays appropriately for the intended sterilization process (e.g. in Aesculap sterile containers).
- ▶ Ensure that the packaging provides sufficient protection against recontamination of the product during storage.

7.9 Steam sterilization

Note

To avoid breakage due to stress crack corrosion, sterilize the product with the lock fully open or locked no further than on the first ratchet tooth.

- ▶ Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- ▶ Validated sterilization process
 - Steam sterilization using fractional vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractional vacuum process at 134 °C, holding time 18 minutes for prion inactivation
- ▶ When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

7.10 Storage

- ▶ Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

8. Technical Service

WARNING

Risk of injury and/or malfunction!

- ▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95-1602

Fax: +49 (7461) 16-5621

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

9. Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

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