

NasoSafe is an innovative Nasopharyngeal Airway. The swivel safety grip eliminates the risk of the airway from being drawn down the nasal passage whilst providing maximum comfort for the patient.

NasoSafe®



Patient Comfort

The safety grip can be rotated through 360° to place the airway in the most convenient and comfortable position without blocking the free nostril.

Allows Access

NasoSafe is ideal when the oral passage is not accessible and allows for nasotracheal suctioning.

Reduced Risk of Injury

The rounded edge of the tip combined with the angle cut design helps reduce the risk of injury during intubation.

Size Colour Coding

Available in 11 sizes with colour coded swivel grips, that co-ordinate with suction catheter sizes.

Swivel Safety Grip

The NasoSafe swivel grip prevents the nasopharyngeal airway from being drawn into the nasal passage without the need for an additional fixation device, such as a safety pin.

Better Tolerance

NasoSafe is better tolerated by awake patients than oropharyngeal airways.



NasoSafe is best inserted when the patient is in the supine position. The airway should be lubricated with a water soluble lubricant prior to insertion.

The rounded bevelled tip reduces mucosal wall trauma and guides the airway along the floor of the nasal passage. When correctly in place the tip of the airway should be visible behind the uvula.

		mm	fr		
		ID	OD		
038-95-004	NasoSafe Size 4 - Dark Green	4.0mm	20fr	●	10
038-95-045	NasoSafe Size 4.5 - Dark Green	4.5mm	22fr	●	10
038-95-005	NasoSafe Size 5 - Black	5.0mm	24fr	●	10
038-95-055	NasoSafe Size 5.5 - Black	5.5mm	26fr	●	10
038-95-006	NasoSafe Size 6 - Green	6.0mm	28fr	●	10
038-95-065	NasoSafe Size 6.5 - Green	6.5mm	28fr	●	10
038-95-007	NasoSafe Size 7 - Orange	7.0mm	30fr	●	10
038-95-075	NasoSafe Size 7.5 - Orange	7.5mm	32fr	●	10
038-95-008	NasoSafe Size 8 - Dark Red	8.0mm	34fr	●	10
038-95-085	NasoSafe Size 8.5 - Dark Red	8.5mm	36fr	●	10
038-95-009	NasoSafe Size 9 - Purple	9.0mm	38fr	●	10

NasoClear is a simple and effective airway adjunct designed for easy insertion and maximum patient compliance.



In-Built Safety

NasoClear has an integrated connector designed to prevent the tube from travelling down the nasal passage and removes the need to use a safety pin, eliminating the risk of needle stick injury.

Clear Construction

Provides visual confirmation before insertion that there are no blockages.

Reduced Risk of Injury

Rounded bevelled tip reduces mucosal wall trauma and guides the airway along the floor of the nasal passage.

Patient Comfort

NasoClear is better tolerated by awake patients than oropharyngeal airways.


Flared Nasal Opening

The large flared opening prevents the nasopharyngeal airway from physically travelling down the nasal passage and allows for comfortable positioning without obstructing the other nostril.


Patient Fit

Available in 11 sizes for optimal patient fit.


Nasopharyngeal Airway Insertion Technique

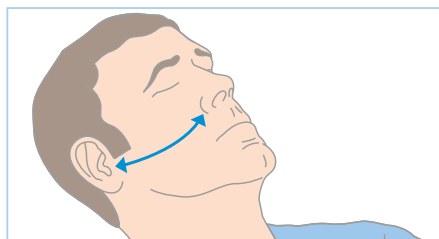

The length of the nasal airway can be estimated by the distance from the patient's nostril to the earlobe, or the angle of the jaw.

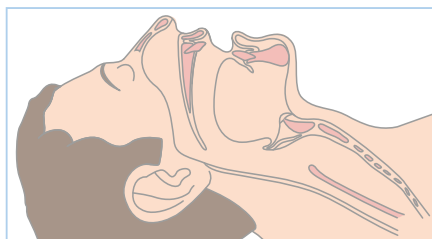

The nasal airway is best inserted when the patient is in the supine position. Lubricate the airway prior to insertion with a water soluble lubricant.

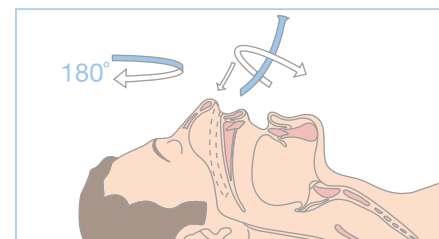

Insert with the curve facing the opposing direction and follow the nasal passage whilst turning the nasopharyngeal airway through 180°

		mm	fr	
		ID	OD	
038-95-004C	NasoClear Size 4	4.0mm	20fr	10
038-95-045C	NasoClear Size 4.5	4.5mm	22fr	10
038-95-005C	NasoClear Size 5	5.0mm	24fr	10
038-95-055C	NasoClear Size 5.5	5.5mm	26fr	10
038-95-006C	NasoClear Size 6	6.0mm	28fr	10
038-95-065C	NasoClear Size 6.5	6.5mm	28fr	10
038-95-007C	NasoClear Size 7	7.0mm	30fr	10
038-95-075C	NasoClear Size 7.5	7.5mm	32fr	10
038-95-008C	NasoClear Size 8	8.0mm	34fr	10
038-95-085C	NasoClear Size 8.5	8.5mm	36fr	10
038-95-009C	NasoClear Size 9	9.0mm	38fr	10