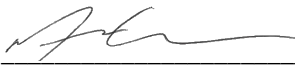
Approved by Product Manager

Print Name: Matthew Crombeen

Signature:  Date: Apr 23, 2021

Approved by Quality and Regulatory Manager

Print Name: Maria Rejak

Signature: 
Maria Rejak (Apr 26, 2021 08:23 EDT) Date: Apr 26, 2021

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STERILE R
SOLUTION

100

Flush Syringes – This device is intended for use in flushing compatible intravenous administration sets and indwelling intravenous access devices.
(Do not place on sterile field.)

Rx ONLY CAUTION: See package for use. **LATEX** Not Made with Natural Rubber. Avoid sunlight. Keep dry. Single Use. Do not use if package is damaged or tampered. Avoid Freezing.

Manufactured in the USA
Trudell Healthcare Solutions Inc.
738 Burnhamway Drive, London, ON, Canada N5Y 5J7
800-265-5494

A4881_R1

www.trudellhs.com

LOT xxxxxxxx
YYYYMMDD

- DIRECTIONS FOR USE:**
1. The fluid and fluid path are sterile, non-pyrogenic and non-toxic. Do not place on sterile field.
 2. Do not use if the syringe or its cap is damaged or displaced or if any leakage is evident. Do not use if solution is hazy, cloudy, discolored or contains a precipitate.
 3. Remove syringe from packaging.
 4. Remove syringe cap.
 5. If required, attach needle or cannula to syringe.
 6. Hold syringe upright and expel air bubble. Syringe is now ready for use.
 7. Use in accordance with recommendations of manufacturer of I.V. access device. Use aseptic technique according to institutional policy. Ensure injected volume is equivalent to access device.
 8. After use, confirm patency of the access device.
 9. For single use only. Discard syringe and any unused portion. Do not reuse. Osmolarity is 0.31 mOsm/mL (calc).

INDICATIONS & USAGE: 0.9% Sodium Chloride I.V. Flush Syringe is intended for use in flushing compatible intravenous administration sets and indwelling intravenous access devices. Use according to the recommendations for the appropriate device.

CONTRAINDICATIONS: None known. **WARNINGS:** Do not use if fluid is cloudy or has any type of suspended particulate matter. Do not use if the syringe cap is damaged in any way. **PRECAUTIONS:** Do not reuse. For single use only. Discard any unused portion. Verify the expiration date on the product label. Avoid contact between this solution and incompatible drug products. Consult drug compatibility literature.

ADVERSE REACTIONS: None known at these quantities.

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www.trudellhs.com

4645-5494
Trudell Healthcare Solutions Inc.
738 Burnhamway Drive, London, ON, Canada N5Y 5J7

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Final Audit Report

2021-04-26

| | |
|-----------------|--|
| Created: | 2021-04-23 |
| By: | Agnes Dusza (adusza@trudellhs.com) |
| Status: | Signed |
| Transaction ID: | CBJCHBCAABAAZwJMDq1sqSAa-YFXjb6PNe7xXcWb0bbz |

"T161203-BP_TruParam_InnerCarton_A6881,R1_April 23, 2021" History

-  Document created by Agnes Dusza (adusza@trudellhs.com)
2021-04-23 - 8:43:47 PM GMT- IP address: 72.138.12.196
-  Document emailed to Matthew Crombeen (mcrombeen@trudellhs.com) for signature
2021-04-23 - 8:45:25 PM GMT
-  Email viewed by Matthew Crombeen (mcrombeen@trudellhs.com)
2021-04-23 - 9:24:28 PM GMT- IP address: 72.138.12.196
-  Document e-signed by Matthew Crombeen (mcrombeen@trudellhs.com)
Signature Date: 2021-04-23 - 9:25:02 PM GMT - Time Source: server- IP address: 72.138.12.196
-  Document emailed to Maria Rejak (mrejak@trudellhs.com) for signature
2021-04-23 - 9:25:04 PM GMT
-  Email viewed by Maria Rejak (mrejak@trudellhs.com)
2021-04-26 - 12:23:26 PM GMT- IP address: 72.138.12.196
-  Document e-signed by Maria Rejak (mrejak@trudellhs.com)
Signature Date: 2021-04-26 - 12:23:51 PM GMT - Time Source: server- IP address: 72.138.12.196
-  Agreement completed.
2021-04-26 - 12:23:51 PM GMT