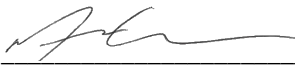
Approved by Product Manager

Print Name: Matthew Crombeen

Signature:  Date: Apr 23, 2021

Approved by Quality and Regulatory Manager

Print Name: Maria Rejak

Signature: 
Maria Rejak (Apr 26, 2021 08:24 EDT) Date: Apr 26, 2021

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100

STERILE FLUID PATH

Flush Syringes – This device is intended for use in flushing compatible intravenous administration sets and indwelling intravenous access devices.

(Do not place on sterile field.)

RX ONLY CAUTION: See Package Insert for Use Only
Not Made with Natural Rubber
Avoid sunlight
Keep dry
Single Use
Do not use if package is damaged or freezing

LOT xxxxxxxx
YYMMDD
YYYYMMDD

Manufactured in the USA
Trudell Healthcare Solutions Inc.
738 Burnhamy Drive, London, ON, Canada N5V 5J7
800-265-5494

A4884.R1

www.trudellhs.com

- DIRECTIONS FOR USE:**
1. The fluid and fluid path are sterile, non-pyrogenic and non-toxic. Do not place on sterile field.
 2. Do not use if the syringe or its cap is damaged or displaced or if any leakage is evident. Do not use if solution is hazy, cloudy, discolored or contains a precipitate.
 3. Remove syringe from packaging.
 4. Remove syringe cap.
 5. If required, attach needle or cannula to syringe.
 6. Hold syringe upright and expel air bubble. Syringe is now ready for use.
 7. Use in accordance with recommendations of manufacturer of I.V. access device. Use aseptic technique according to institutional policy. Ensure injected volume is equivalent to access device.
 8. After use, confirm patency of the access device.
 9. For single use only. Discard syringe and any unused portion. Do not reuse. Osmolarity is 0.31 mOsm/mL (calc).

INDICATIONS & USAGE: 0.9% Sodium Chloride I.V. Flush Syringe is intended for use in flushing compatible intravenous administration sets and indwelling intravenous access devices. Use according to the recommendations for the appropriate device.

CONTRAINDICATIONS: None known. **WARNINGS:** Do not use if fluid is cloudy or has any type of suspended particulate matter. Do not use if the syringe cap is damaged in any way. **PRECAUTIONS:** Do not reuse. For single use only. Discard any unused portion. Verify the expiration date on the product label. Avoid contact between this solution and incompatible drug products. Consult drug compatibility literature.

ADVERSE REACTIONS: None known at these quantities.

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Final Audit Report

2021-04-26

Created:	2021-04-23
By:	Agnes Dusza (adusza@trudellhs.com)
Status:	Signed
Transaction ID:	CBJCHBCAABAAVJ6og6oW1yz66hBOSAgiKBH7_anD9DKU

"T161205-BP_TruParam_InnerCarton_A6884,R1_April 23, 2021" History

-  Document created by Agnes Dusza (adusza@trudellhs.com)
2021-04-23 - 8:47:34 PM GMT- IP address: 72.138.12.196
-  Document emailed to Matthew Crombeen (mcrombeen@trudellhs.com) for signature
2021-04-23 - 8:48:11 PM GMT
-  Email viewed by Matthew Crombeen (mcrombeen@trudellhs.com)
2021-04-23 - 9:25:12 PM GMT- IP address: 72.138.12.196
-  Document e-signed by Matthew Crombeen (mcrombeen@trudellhs.com)
Signature Date: 2021-04-23 - 9:25:35 PM GMT - Time Source: server- IP address: 72.138.12.196
-  Document emailed to Maria Rejak (mrejak@trudellhs.com) for signature
2021-04-23 - 9:25:36 PM GMT
-  Email viewed by Maria Rejak (mrejak@trudellhs.com)
2021-04-26 - 12:24:07 PM GMT- IP address: 72.138.12.196
-  Document e-signed by Maria Rejak (mrejak@trudellhs.com)
Signature Date: 2021-04-26 - 12:24:21 PM GMT - Time Source: server- IP address: 72.138.12.196
-  Agreement completed.
2021-04-26 - 12:24:21 PM GMT